

How to connect the App

Note:

Do not use the device unless you have successfully connected the device to your mobile phone through the Android/iOS App and are streaming live video footage.

1

Download the APP

iOS: Search for "bebird" in App Store
Android: Search for "bebird" in Google Play



OR scan the QR code to download the app



2

Open the cover
Power on automatically



Open the cover
Power on automatically



Blue light flashes


3

Open the APP
Connect device WIFI



4

Connect Product Wi-Fi
"bebird-M9-XXXXXX"

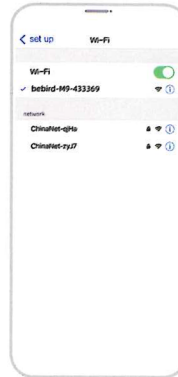


5

Back to bebird APP
Click on "GO" to use it




6

The visual screen

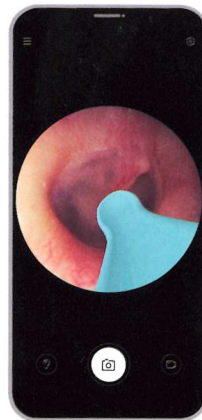

Product Overview

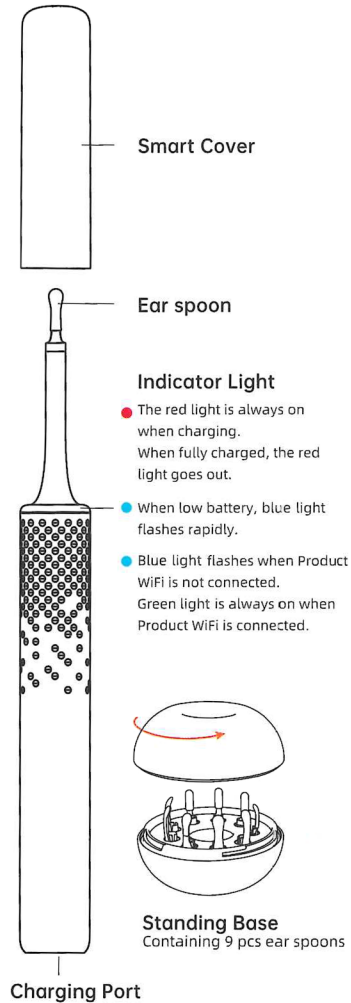


How to Replace Ear Spoon



If the ear spoon is damaged, please stop using it immediately.


Please make sure the ear spoon is secured on the device before use.

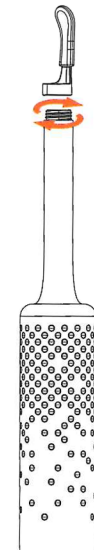

Screw off the ear spoon and replace with a new one.

Please replace the ear tip with a new one every 1-2 months to avoid aging and breakage.

Accessories

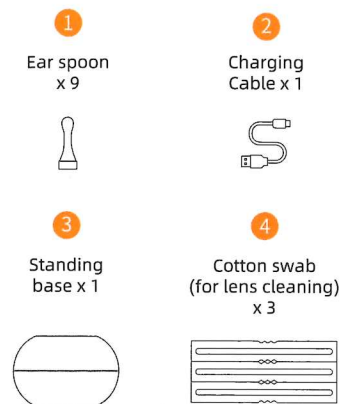

Troubleshooting

Problem	Solution
No response after startup	Please use the incidental charging cable to charge
The LED light of the camera is not on	Interrupt the connection, restart the product to connect again
The app exits quickly	Delete the existing app and scan QR to reinstall it
The indicator light blinks quickly	Please use the incidental charging cable to charge
Abnormal display in app	Interrupt the connection, restart the product to connect again
The Wifi connection was unstable and disconnected in a few seconds	Interrupt the connection, restart the product to connect again
The visual interface blur	Please wipe it with a double-ended cleaning brush or alcohol swab
Disorientation when in use	Please switch the left / right ear mode under the APP interface
If you still have problems with the above procedures, please contact our Customer Service Representatives directly for further assistance.	

Product Parameters

Brand	bebird
Product Name	Smart Visual Ear Cleaner
Model	M9 S
Network standard	IEEE 802.11b/g
Antenna	Internal FPC antenna
Working frequency	2.4Ghz
Image transmission rate	20fps
image sensor	CMOS
Working temperature	-10 ~ 50°C
Battery	230mAh lithium battery
Battery life	about 90 minutes
Charging time	about 60 minutes
Input Parameters	5V --- 0.5A
Lens diameter	3.5mm
Optimum focal length	13.5mm
Pixel	10 Mega-pixels
Gyroscope	Gimbale Intelligent Gyroscope

bebird M9 S
Smart Visual Ear Cleaner



Attention

- When cleaning the lens, wipe it carefully with an alcohol cotton swab.
- Do not use the device while you are moving. Use in a safe place to avoid being bumped by others.
- This product is unsuitable for children under the age of 3.
- Children should not use this device unsupervised by an adult, to avoid accidental harm.
- Do not drop or submerge the device in liquid, to avoid damage.
- This product has a built-in lithium battery. If not used frequently, it is recommended to charge the battery once a month.
- Do not expose the device to direct sunshine. Also keep the accessories out of direct sunshine to avoid softening.
- During use, the device may warm up slightly (up to 35 degrees). This is normal and you can continue to use the device.
- When using the device on a child, for the best experience, keep the child in a sitting position and use the device horizontally.

Hazardous Substance

Part name	Hazardous Material					
	(Pb)	(Hg)	(Cd)	(Cr(VI))	(PBB)	(PBDE)
Printed Circuit Components	×	○	○	○	○	○
Case	○	○	○	○	○	○
Plastic rubber accessory	○	○	○	○	○	○

This form is formulated in accordance with SJ/T 11364
 ○: represent the hazardous material ratio is below GB/T26572-2011
 ×: represent the hazardous material ratio is over GB/T26572-2011

Warranty

User info

User name:

Address:

Telephone number:

Seller:

Telephone number:

▲ Note: Do not use the charging equipment with output voltage exceeding 5V