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Adhesive for hearing tubes

Extended purpose

Primer for the bonding of acrylic or PVC plastics with silicone materials

Indication for use

For a firm bonding between a behind-the-ear hearing device and a silicone earmold, to ideally transmit sound waves to the ear drum of the person wearing the hearing device.

Contraindication

The product may not be used if the patient is allergic to one of the ingredients. There are no side-effects to be expected from this medical device if processed and used properly. However, immune reactions (e.g. allergies) or local disorders of sensation (e.g. irritation in the auditory canal) cannot be ruled out in principle. If you become aware of adverse side-effects, also in cases of doubt, please be sure to advise us in any case and describe the circumstances and symptoms as accurately as possible. We will investigate any notice.

Target Patient Group

Patients who need ear care.

Intended users

Trained staff (e. g. earmould technicians)

Product description

Glusil is a transparent, single-component bonding agent to be used on PVC hearing tubes and acrylic earmolds (FotoTec®/Fotoplast®) to create a secure and permanent bonding with the silicone materials.

Technical data

Technical datasheet available on request.

Processing

Grease the locations to be bonded (roughen the surface if necessary) and subsequently apply a thin and bubble-free layer of Glusil; let it dry for 10 minutes. The silicone lacquer or SK 87 silicone glue can then be applied in accordance with the corresponding instructions. Information on processing can be obtained from the corresponding instructions.

Caution

- The product is exclusively intended for the above-mentioned range of use and shall only be used by professionally qualified and instructed persons.
- Wear protective clothing!
- Use a suitable extraction system when working.
- Immediately close the bottle after use.

Single-use advice

The cured primer cannot be used for any further priming due to its physical characteristics.

Disposal

Disposal of contents / container in accordance with local / regional / national / international regulations and safety data sheet.

Hazard identification

Hazard and safety notes can be found on the product label and the corresponding safety data sheet.

Serious incidents

All serious incidents occurring in relation to the product shall be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is settled.

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Die Dreve Otoplastik GmbH haftet nicht für Schäden, die durch fehlerhafte Anwendung des Produktes hervorgerufen werden.
Dreve Otoplastik GmbH is not liable for any damages caused by improper application of the product.
Dreve Otoplastik GmbH no se hace responsable de los daños causados por una aplicación incorrecta del producto.