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Intended use
Adhesive for hearing tubes
Extended purpose
For glueing acoustic tubings into silicone earmolds

Indication for use
For a firm connection between a behind-the-ear hearing device and a silicone earmould to optimally transmit the sound waves to the ear drum of the person wearing the hearing device.

Contraindication
The product may not be used if the patient is allergic to one of the ingredients. There are no side-effects to be expected from this medical device if processed and used properly. However, immune reactions (e.g. allergies) or local disorders of sensation (e.g. irritation in the auditory canal) cannot be ruled out in principle. If you become aware of adverse side-effects, also in cases of doubt, please be sure to advise us in any case and describe the circumstances and symptoms as accurately as possible. We will investigate any notice.

Target Patient Group
Patients who need ear care.

Intended users
Trained staff (e.g. earmould technicians)

Product description
SK 87 Silicone glue is a special glue for silicone. It can be used to glue silicone tubes or Glusil-coated tubes into silicone earmoulds.

Technical data
Technical datasheet available on request.

Processing
Degrease the points to be glued with the cleaning fluid and then apply the SK 87 Silicone glue. PVC hearing tubes must first be coated with Glusil, this step is not required for silicone tubes. To glue, coat the hearing tube with SK 87 Silicone glue and pull it into the earmold. Curing: Glusil approx. 10 min. SK 87 Silicone glue approx. 30 min.

- Caution**
- The product is exclusively intended for the above-mentioned range of use and shall only be used by professionally qualified and instructed persons.
 - Wear protective clothing! Set material is chemically resistant. Marks on your clothing cannot be removed.
 - Use a suitable extraction system when working.
 - Keep the tube securely closed in the original packaging after use

Note on one-time use
Due to the physical characteristics, the cured silicone glue cannot be used for further coating.

Disposal
Disposal of contents/container in accordance with local/regional/national/international regulations and safety data sheet.

Hazard identification
Hazard and safety notes can be found on the product label and the corresponding safety data sheet.

Serious incidents
All serious incidents occurring in relation to the product shall be reported to the manufacturer and to the competent authority of the Member State in which the user and/ or patient is settled.

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Die Dreve Otoplastik GmbH haftet nicht für Schäden, die durch fehlerhafte Anwendung des Produktes hervorgerufen werden.
Dreve Otoplastik GmbH is not liable for any damages caused by improper application of the product.
Dreve Otoplastik GmbH no se hace responsable de los daños causados por una aplicación incorrecta del producto.