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Indicatie

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Accessories for ear impressions

Indication for use

Together with ear impression material for reproduction of the outer ear canal of a patient.

Contraindication

The product may not be used if the patient is allergic to one of the ingredients. There are no side-effects to be expected from this medical device if processed and used properly. However, immune reactions (e.g. allergies) or local disorders of sensation (e.g. irritation in the auditory canal) cannot be ruled out in principle. If you become aware of adverse side-effects, also in cases of doubt, please be sure to advise us in any case and describe the circumstances and symptoms as accurately as possible. We will investigate any notice.

Target Patient Group

Patients who need ear care.

Intended users

ENT specialist, hearing aid audiologist

Product description

With the Dreve impression syringe made of plastics, silicone moulding material of kneadable consistency can be professionally processed to yield perfect results without bubble inclusions. Thanks to the double-piston functionality, easy filling and uniform, smooth injection is possible.

Application

- 1. Fill homogeneously mixed non-sticky impression silicone in the big cylinder part of the impression syringe.
- 2. With the help of the piston the impression material has to be pressed into the narrow cylinder. The part of handle has to be placed into the hindmost position
- 3. Slightly and evenly inject the impression silicone into the auditory canal and then into the concha.

Disposal

Disposal of contents / container in accordance with local / regional / national / international regulations.

Cleaning

After the impression material has cured disperse the impression syringe. Remove rests of the impression silicone. For cleaning and disinfection of the impression syringe use commercially available disinfecting agents which do not contain solvents like acetone. After the prescribed time dry the syringe with cleaned cloths.

Serious incidents

All serious incidents occurring in relation to the product shall be reported to the manufacturer and to the competent authority of the Member State in which the user and/ or patient is settled.

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Die Dreve Otoplastik GmbH haftet nicht für Schäden, die durch fehlerhafte Anwendung des Produktes hervorgerufen werden.
Dreve Otoplastik GmbH is not liable for any damages caused by improper application of the product.
Dreve Otoplastik GmbH no se hace responsable de los daños causados por una aplicación incorrecta del producto.