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Caution!

The impression of the outer ear canal may only be taken with impression material by persons with appropriate qualifications. The processing information and precautions which are set out in these directions for use must be observed under all circumstances. Failure to observe these may result in irreparable damage to the ear or the eardrum. If damage or undesirable side effects occur despite the warnings, we request that you report this with a detailed description of your procedure and the damage to or side effects for the patient.

Intended use

A-silicone for the production of earmolds

Extended intended use

A-silicone for direct fitting of earmolds

Indication for use

For direct fitting of individual earmolds for a hearing aid or as a protective earmold.

Contraindication

The product may not be used if the patient is allergic to one of the ingredients. There are no side-effects to be expected from this medical device if processed and used properly. However, immune reactions (e. g. allergies) or local disorders of sensation (e. g. irritation in the auditory canal) cannot be ruled out in principle. If you become aware of adverse side-effects, also in cases of doubt, please be sure to advise us in any case and describe the circumstances and symptoms as accurately as possible. We will investigate any notice.

Patient groups

Otoform® Protect and Otopren must not be used for patients suffering from a perforated eardrum or injuries to the ear canal.

Target Patient Group

Patients who need ear care.

Intended users

ENT specialist, hearing aid audiologist

Product description

Otoform® Protect consists of addition-curing silicone for the production of BTE and splash water protection earmolds directly in the ear. Otoform® Protect is a kneadable impression silicone, non-sticky and smooth, with a slightly pressure-forming consistency. Earmolds manufactured from Otoform® Protect float on the water surface.

Otopren consists of addition-curing silicone for the production of earmolds directly in the ear. Otopren is soft, stable/thixotropic in consistency. It is processed in a double cartridge in a mixing ratio of 1:1. Thanks to the platinum hardener system, Otoform® Protect and Otopren are practically shrink-proof and physiologically well tolerated.

Batch number / use-by date

The batch number and the use-by date are located on both the outer packaging and the product. When submitting a complaint about a product, please always quote the batch number. Do not use the product after expiration of the use-by date.

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Processing

The ear canal and the eardrum must be examined before the impression is taken. No impression may be taken in the event of abnormal findings such as inflammation or perforation of the ear drum. Cleaning of the ear canal and removal of hairs by an ENT doctor is required in the event of significant earwax deposits and hairs in the ear canal. An impression pad must be placed at the end of the outer ear canal before the eardrum before the impression is taken in order to protect the eardrum.

Double cartridge: The impression material from the double cartridge is automatically mixed at a 1:1 ratio. For this, place the double cartridge into the injector. Remove the cap and dispense a little material from the double cartridge in order to ensure that the two components come out evenly. Now place a mixing canula on the double cartridge. Check whether the material is dispensed evenly and mixed from the canula. To do this, dispense a little material again before beginning taking the actual impression. The impression material is slowly applied into the ear canal and the concha. Please take care to ensure that the tip of the mixing canula is always in direct contact with the impression mass which has been dispensed. After applying the impression material, please leave the mixing canula on the double cartridge as a cap and replace it with a new mixing canula for the next impression.

Tubs: Remove equal parts of the two components of the impression material from the containers with dosing spoons and press them onto a mixing tray. Now knead both components quickly with a spatula until a uniform color is obtained. For hygiene reasons, kneading by hand is only recommended with gloves (no latex gloves!). The kneaded mass is now filled into an impression syringe. The impression material has to be applied slowly into the auditory canal and the concha. Please take care to ensure that the tip of the impression syringe is always in direct contact with the impression material during application. If necessary, you can form a handle with the soft stuff material. Carefully close the tubs after use.

Once the material has hardened, the earmold can be carefully removed from the ear. The ear canal must be examined again after the impression has been taken.

Please find processing times in “technical data” overview.

Post-processing

Process the earmold with milling cutters for soft materials. The earmolds can then be lacquered. To upgrade the surfaces, we recommend silicone lacquers, e.g. Lack B eco.

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Cleaning / disinfecting

For thorough cleaning and disinfection, we recommend our OtoVita® care series.

Precautions

- The product is exclusively intended for the above-entioned range of use and shall only be used by professionally qualified and instructed persons.
- Pre-convey the material before attaching the mixing canula.
- Polymerisation, referring to the hardening of addition curing silicones, may be inhibited by cerumen, latex gloves, creams, plastics, cleaning agents, etc. For this reason, we recommend that you wear commercially available polyethylene gloves.
- The setting times can be increased by lower temperatures or reduced by higher temperatures.
- Wear protective clothing! Set material is chemically resistant. Marks on your clothing cannot be removed.
- Wear suitable personal protective equipment (protective gloves, safety goggles, face mask) during reworking of the cured material.
- After use, leave the mixing canula on the double cartridge as a cap.
- Never dispense into a mixing canula which has hardened.

Keep away from children!

Note for one-time use

Due to the physical properties, the cured earmold silicone cannot be used several times.

Disposal

Disposal of contents / container in accordance with local / regional / national / international regulations and safety data sheet.

Hazard identification

Hazard and safety notes can be found on the product label and the corresponding safety data sheet.

Serious incidents

All serious incidents occurring in relation to the product shall be reported to the manufacturer and to the competent authority of the Member State in which the user and / or patient is settled.

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