General Information

Regular use of these **ER-20 High-Fidelity Earplugs** while practicing, performing and listening to music will protect you from the cumulative effects of overexposure to loud sound throughout your life. Overexposure to loud sounds can harm the tiny hair cells in the inner ear, which can lead to hearing loss and ringing in the ears. Foam earplugs muffle sound, but ER-20 earplugs use a unique acoustic construction that produces almost equal sound reduction (20 dB) at all frequencies without changing the clarity of speech or the richness of music. Music and speech stay clear, just quieter, like turning down the volume. ER-20 earplugs are reusable.

Insertion

- Before inserting earplugs, make sure they are clean.
- Grasp the end cap between thumb and forefinger.
- To ease insertion, pull the top of the ear outward and upward (see illustration).
- Gently push and twist the earplug until the eartip seals in the auditory canal.
- Moistening the eartip may ease insertion.

Removal

- Remove with a slow twisting motion to gradually break the seal and avoid discomfort.
- After removal, place earplugs in a clean protective case.

Cleaning

- Do not take earplugs apart.
- Clean after each use.
- Wipe clean with a damp cloth as needed.
- Do not immerse earplugs in water.
- Do not clean with harsh chemicals.
- Alcohol is not recommended.

Storage

- Always store earplugs in a clean protective case.
- Do not store other objects, except a neck cord, in the earplug case.

Warnings

- These earplugs comply with EN 352-2 standard of measurement.
- The triple-flange eartip must be fitted, adjusted and maintained properly, i.e. seal the ear completely, to achieve the expected attenuation and hearing protection.
- Use earplugs continuously when noise levels are potentially harmful.
- Make certain that the earplugs provide adequate protection for the noisy environment.
- Regularly inspect the earplugs to assure their continued serviceability.
- Never remove the end cap while the earplugs are in use.
- Failure to follow these warnings may severely reduce the amount of hearing protection provided by the earplugs.
- This product may be adversely affected by certain chemical substances.

For additional information, contact

egger Otoplastik + Labortechnik GmbH, Aybühlweg 59, 87439 Kempten/Germany, www.egger-gehoerschutz.com or the manufacturer
Etymotic Research, Inc., 61 Martin Lane, Elk Grove Village, IL 60007, U.S., www.etymotic.com.

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egger Otoplastik + Labortechnik GmbH, Aybühlweg 59, 87439 Kempten/Germany, www.egger-gehoerschutz.com o el fabricante