## egger LP/H

klar, rosa-transparent, beige, flesh, rot, blau

## egger LP/HS

klar, rosa-transparent

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7909207-2010-08

## General product information

LP/H material can be processed in one of the following egger EL devices:

egger EL 1 REF 51100 egger EL 2 REF 51200  
egger EL 1 plus N<sub>2</sub> REF 51110

Always observe the user instructions of the device you are using and the polymerization times indicated below.

**Delivery units:** See catalog

**Expiry date and charge:** See bottom of the bottle or crimped end of tube.

**Storage:** The material should be stored in a cool and dry place, away from foodstuff. Keep container always closed. Protect from light.

### Notes:

- Close the container after use.
- In case of air bubbles in the material heat up the open container (but covered with the lid) to a temperature of approx. 50 °C (122 °F) for 15 to 30 minutes.
- Avoid direct skin contact with the liquid material! Hands should be protected with appropriate gloves before working with the material and finishing the blanks.

### Notes for processing:

Before working with the material, it must have a temperature of approximately 23 °C (73.4 °F) at a relative air humidity of 50% ± 5%.

## Processing

### Manufacturing of hard BTE earmolds or ITE shells

1. Fill the LP/H or LP/HS material into the negative mold made of duplicating material (REF 409 ff).
2. Make sure that the negative mold is completely filled and bubble free.
3. Cover it with black foil (REF 53200 or REF 53100)
4. Put the filled negative mold into the UV-polymerization unit and let it polymerize according to the following polymerization times.  
**No gas flow is required if you work with the UV-polymerization unit EL 1 plus N<sub>2</sub>.**

LP/H and LP/HS transparent and pink-transparent	
EL 1	2 minutes 30 seconds to 3 minutes
EL 1 plus N <sub>2</sub>	2 minutes 30 seconds to 3 minutes (without gas)
EL 2	3 minutes to 3 minutes 30 seconds

LP/H beige	
EL 1	2 minutes 30 seconds to 3 minutes
EL 1 plus N <sub>2</sub>	2 minutes 30 seconds to 3 minutes (without gas)
EL 2	3 minutes to 3 minutes 30 seconds

LP/H flesh	
EL 1	1 minute 10 seconds to 1 minute 20 seconds
EL 1 plus N <sub>2</sub>	1 minute 10 seconds to 1 minute 20 seconds (without gas)
EL 2	1 minute 20 seconds to 1 minute 30 seconds

LP/H red and blue	
EL 1	30 seconds
EL 1 plus N <sub>2</sub>	30 seconds (without gas)
EL 2	1 minute to 1 minute 10 seconds

### ATTENTION:

**The mentioned polymerization times are guide values, they may vary depending on the size of the earmold and the transparency of the negative mold.**

5. Remove the black foil after polymerization.
6. Pour the liquid material into a container that is impervious to light. **This material can be reused.**
7. After having completely poured out the liquid material, put the negative mold and the blank but without the black foil into the UV-polymerization unit again.  
**We recommend placing ITE shells in the unit with the opening facing downward.**  
No gas flow is required if you work with the UV-polymerization unit EL 1 plus N<sub>2</sub>.

LP/H and LP/HS transparent and pink-transparent	
EL 1	1 minute to 2 minutes
EL 1 plus N <sub>2</sub>	1 minute to 2 minutes (without gas)
EL 2	2 minutes to 3 minutes

LP/H beige, flesh, red and blue	
EL 1	2 minutes to 4 minutes
EL 1 plus N <sub>2</sub>	2 minutes to 4 minutes (without gas)
EL 2	4 minutes to 5 minutes

8. Take the blank out of the negative mold and put it into a transparent container (REF 50107) filled with LP-Cleaner (REF 313 ff). Then put it into a UV-polymerization unit for the second polymerization. **If you work under gas, put the blank into a transparent box with cover (REF 51114).**

EL 1	5 minutes to 6 minutes
EL 1 plus N <sub>2</sub>	5 minutes to 6 minutes with gas
EL 2	7 minutes to 8 minutes

9. Then remove the blank. If you worked with LP-Cleaner (REF 313 ff) rinse the blank under running water.

### Treatment and lacquering:

1. The blank can now be shaped with the special egger carbide cutters.
2. Before lacquering the surface, roughen it with abrasive paper, grit 180, (REF 40704) then clean and degrease it with surface cleaner (REF 3160 ff). Afterwards you can coat the earmold/ITE shell with the LP/H lacquer (REF 3060 f), LP/H lacquer antibac (REF 3062 f) or LP/W lacquer (REF 3070 f).